

- ALCOHOL INFLUENCE REPORT FORM

Defendant (First name)		(Initial)	(Last name)	Case Number	Sequential File No.
Age	Sex	Weight	Eyes	Arrested by	

CHEMICAL BREATH TEST INFORMATION

Instrument: Breathalyzer, Model 900 Breathalyzer, Model 900A Dominador Albreath

	Instrument Serial Number	Ampoule Control Lot Number	Purge % Results	Samples Taken Date / Time	Blood Alcohol % Results
Test #1					
Test #2					
Test #3					
Test #4					

BREATHALYZER CHECK LIST - N.J.A.C. 13:51-3.6(a)2

Mark all applicable boxes with an "X" or a check mark.

Set Up Phase Verify Power switch is turned "On". If the power switch is in the "Off" position, turn the switch "On".

TESTS

#1	#2	#3	#4	
<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	Preparation Phase
<input type="checkbox"/>	N/A*	N/A*	N/A*	1. Instrument temperature: Verified temperature, thermometer reached 50°C, plus or minus 3°C.
<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	2. Reference Ampoule: Gauged; Inserted in left hand holder.
<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	3. Test Ampoule: Gauged; Opened; Verified volume; Inserted in right hand holder.
<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	4. Bubbler: Inserted into test ampoule; Connected to outlet.
<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	5. Light turned on; Instrument balanced, Blood Alcohol Pointer set on Start Line.
				Purge Phase
<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	6. Turned control knob to the "Take" position; Flushed; Turned control knob to the "Analyze" position.
<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	7. Purge: When red empty signal appeared, waited 90 seconds; Light turned on; Instrument balanced.
<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	8. Purge Result Recorded.
<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	9. Blood Alcohol Pointer set on Start Line.
				Analysis Phase
<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	10. New mouthpiece inserted in breath tube.
<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	11. Take Breath Sample: Turned control knob to the "Take" position; Took breath sample from defendant; Turned control knob to the "Analyze" position.
<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	12. Date & Time Recorded.
<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	13. When red empty signal appeared, waited 90 seconds; Light turned on; Instrument balanced.
<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	14. Breath Test Result Recorded.
				Second or subsequent breath tests
<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	15. Test ampoule removed; properly disposed. Power switch remained "On". Return to Step 1, start next breath test.

*Steps marked with an "N/A" are only performed on the first breath test, and do not have to be repeated on any subsequent breath tests.

Breath Test Operator _____

Copy Given to Subject _____

Rank _____ Signature _____ Badge # _____ Date _____ Time _____